BUG BITE thing®

INSECT BITE + STING SUCTION TOOL


INSTANT RELIEF – NO CHEMICALS – REUSABLE

HOW TO USE THE BUG BITE THING®

ILLUSTRATIONS FOR USE

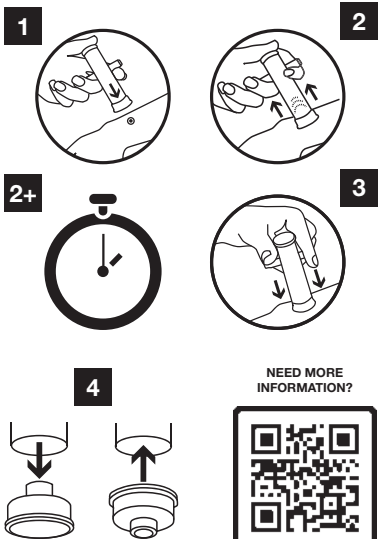NEED MORE INFORMATION?

SCAN ME

REV JFUBBT_2024_04

Bug Bite Thing Europe Aps
Agern Allé 5A, DK-2970 Horsholm

EN

HOW TO USE BUG BITE THING®

INSTRUCTION FOR USE

INTENDED PURPOSE
BUG BITE THING® device is a suction tool intended to alleviate symptoms such as itching, swelling and pain from bug bites and stings.

When you remove the irritant, your body stops producing the reaction that causes itching and swelling.

BUG BITE THING® works on bites and stings from biting insects that leave an irritant just below the surface of the skin. For best result use as quickly as possible after a bite or sting.

FOR FIRST AID ONLY!

In the event of an allergic reaction or severe bite, seek medical attention.

BUG BITE THING® contains no chemicals or creams. Safe for all ages. Children should only use the device with adult supervision.

STEP (1)

Pull the handles down. Place the device over the bug bite or sting area.

STEP (2)

Pull the handles up slightly until you feel suction.

STEP (2+)

Hold for 10-20 seconds.

STEP (3)

Push down on handles to release suction.

REPEAT STEPS (1-3)

2-3 TIMES

REVERSIBLE CAP (4)

Reverse the cap to reveal a smaller area of suction for fingers or toes.

CHILDREN OR SENSITIVE SKIN

Pull handles halfway up or less. Hold for 5-10 seconds. Children or area of sensitive skin (face, neck, or ear) require less suction to remove the irritant due to their thinner skin.

STINGER SCRAPER

Use the side of the handle to scrape out the bee stinger left in the skin.

CLEANING

Wash the removable end cap with soapy water or an alcohol wipe between, and after use.

THE PERFORMANCE CHARACTERISTICS OF THE DEVICE

BUG BITE THING® can be re-used as long as a vacuum can be generated.

WARNING!

NOT FOR USE BY:

Persons with reduced blood clotting capability, such as patients with hemophilia or patients in anticoagulation therapy, persons with known vasculitis such as Schönlein Henoch's purpura or persons with fragile capillaries such as Ehlers-Danlos syndrome - persons with bullous skin disease such as Pemphigoids. Do not use on infected, ulcerated skin or skin with significant eczema.

The device can leave temporary marks/bruises.

Choking hazard: Contains smaller parts.

STOP the suction if you experience any pain.

Please report serious incidents directly to the legal manufacturer of this medical device and the competent Health Authority: Medicines and Healthcare products Regulatory Agency (MHRA), www.mhra.gov.uk

DESIGNED AND MADE IN DENMARK

INSTANT RELIEF – NO CHEMICALS – REUSABLE

LOT ENGRAVED ON THE DEVICE. LOT ENGRAVED ON THE DEVICE.

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